

yuwell



High Flow Nasal Cannula Oxygen Therapy Device (HF-70A)

YUWELL HF-70A

Excellent Design

Intelligent Technology

Flexible and Convenient

Excellent design




With an innovative air way design, the patient exhalation air could not return to the main host, which make the main host does not need to be sterilized, disposable connector and the humidifier chamber, which prevent cross infection.



7-inch touch screen, easily adjust the setting by finger or pen, even with rubber gloves. Screen shows Flow Rate, Temperature, FiO₂, Treat Timer, Dew-Point Temperature.



Intelligent Technology

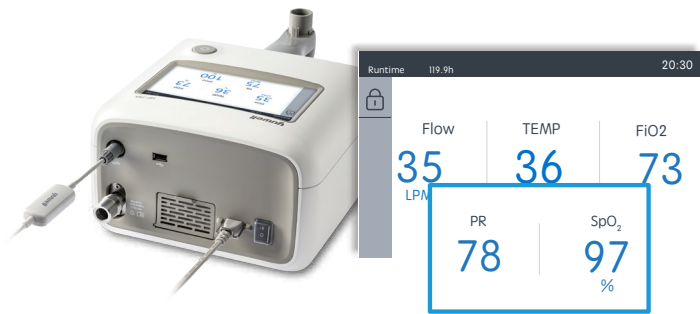

With fingertip pulse oximeter sensor, you can monitor patient's SPO₂ and pulse rate continuously.






The warning and recording function can write down alarm and complete the log. The incident record can write down flow, temperature, oxygen concentration and other specific parameters, finally completing the log.

Intelligent Technology

Real-time display of flow volume, temperature, inhale oxygen concentration (FiO_2), to maintain the stable therapy.

With automatic oxygen blender, the inhale oxygen concentration (FiO_2) can be precisely adjusted from 21% to 100%, step 1%, after setting the FiO_2 , and reach the target data within 2 minutes. Ultrasonic Oxygen Analyzer.

With intelligent temperature control technology, the output temperature can be adjusted in 7 setting (31 °C, 32 °C, 33 °C, 34 °C, 35 °C, 36 °C, 37 °C), You can choose one based on seasons to insure the comfort of the patient.

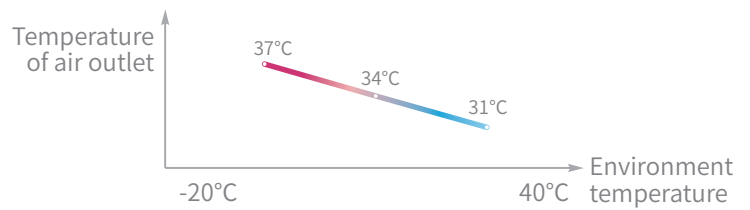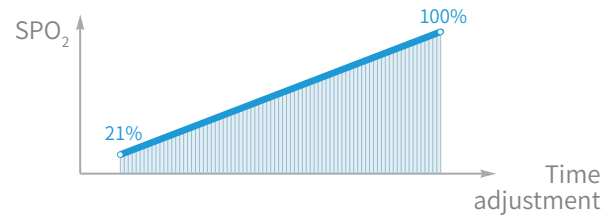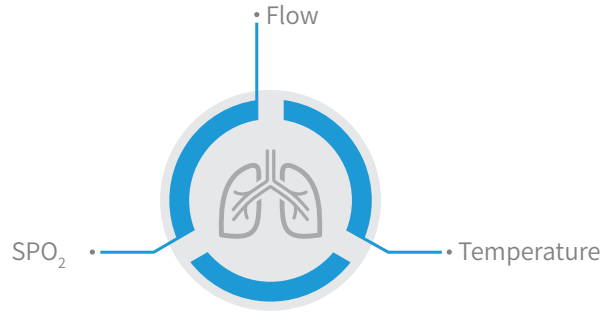

Flexible and convenient

- 8 kinds of warning functions give you ease and reassurance.



Dry humidifier alarm



Tube falling off alarm



Room temperature high alarm



Room temperature low alarm



Oxygen concentration high alarm



Oxygen concentration low alarm



Oxygen input pressure high alarm



Oxygen input pressure low alarm

- With multifunction trolley, the device can be easily moved in the hospital, to meet clinician needs.



Product parameter



Model	HF-70A
Mode	Low flow / High flow
Flow range	2-25LPM/10-70LPM
Temperature	34°C/31°C、34°C、37°C
FiO ₂	21-100%
SPO ₂ sensor	Optional
Screen size	7-inch touch screen
Operating mode	Touch
Disinfection	Free disinfection
Realtime monitoring	Flow, temperature, FiO ₂ , SpO ₂ (optional) , pulse rate (optional)
Alarm	<p style="text-align: center;"> Dry humidifier alarm Tube falling off alarm Room temperature low/high alarm O₂ input pressure low/high alarm Oxygen concentration high/low alarm </p>

YOU WE LOVE LIFE

JIANGSU YUYUE MEDICAL EQUIPMENT & SUPPLY CO., LTD

Sales Center Address: Huanyuan East Road No.1, Xuzhuang Software Park, Nanjing,
Jiangsu Province, P.R. China

Tel: 86-025-8713 6530


Manufacturing Address: No.1 Baisheng Road, Development Zone, Danyang, Jiangsu Province, P.R.China

Tel: 86-0511-8690 0859 86-0511-8690 0860

E-mail: yuwellsales@yuyue.com.cn; sales@yuyue.com.cn

www.yuwell.com

 |  |  |  | Yuwell Global

2020, Yuyue medical, all rights reserved. The "  YUYUE ", yuwell, the yuwell logo and name are trademarked by Jiangsu Yuyue Medical Equipment & Supply Co., Ltd, and may not be distributed, modified, reproduced in whole or in part without the prior. All specifications and product configurations are subject to change without notification.